fidlab™

Body insights
for a better lifestyle.

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This is your kit registration
code. You will need this later to
ship your kit correctly.

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
Table of contents

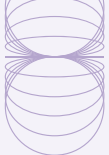1. In this kit	p29
2. Getting ready	p30
Step 1: what not to do	p30
Step 2: register the kit code	p31
Step 3: fill in the online questionnaire	p32
3. Collection instructions	p33
Step 1: preparation	p33
Step 2: collection	p34
Step 3: after collection	p35
4. Shipping	p36
5. Results	p37
6. Payment	p38

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1 afnamebuisje

1 houder (buisbeschermer in hard plastic) in absorberend doekje

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fidlab.be/faq

The Gut Check

The composition of the gut microbiome is considered a reflection of overall health and that of the gut in particular. This test allows you to understand the ecological balance in the gut and its impact on your well-being.

Read more at fidlab.be/gutcheck

A good start

Read through the entire instructions before collection. Not following the instructions correctly may lead to inaccurate or unusable results.

It is essential that the kit code is registered as indicated in the instructions. **An unregistered kit will not provide results.**

Take **at least 30 minutes** to register the kit code, complete the online questionnaire and take the test.

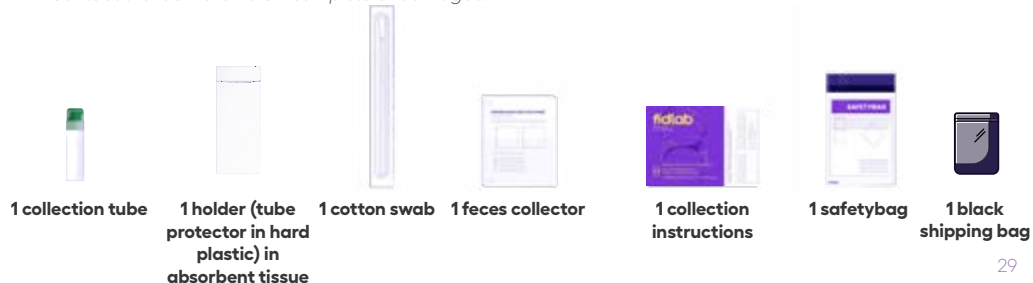
1. In this kit

Remove the content from the box and check that everything is present. Do not throw away the box! You will need it to return the sample after collection.

Content:

- 1 x collection tube with green cap (contains liquid - avoid contact!)
- 1 x holder (tube protector in hard plastic) in absorbent tissue
- 1 x cotton swab
- 1 x feces collector
- 1 x collection instructions (this document)
- 1 x safety bag
- 1 x black shipping bag

Contact the lab if the kit is incomplete or damaged.

2. Getting ready

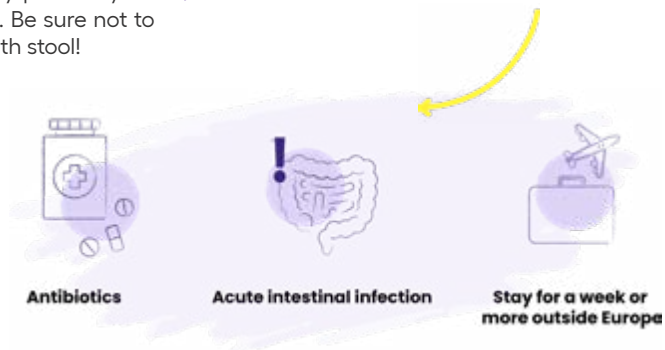
Step 1: what not to do

Avoid contact with the preservative liquid in the tube. The liquid should remain in the tube. In case of contact: rinse well with water.

Make sure your stool **sample is not too large**. A piece of stool maximum the size of a pea is sufficient. More than that may prevent your sample from being examined. Be sure not to stuff the collection tube full with stool!

Avoid the use of **probiotics** within a period of 72h prior to the sample collection.

Avoid the following, **2 weeks prior to sample collection**: antibiotics, acute intestinal infection, stay outside of Europe for a week or more.

Step 2: register the kit code

Before taking the sample, register the kit code online. This kit code starts with the letters MPAA followed by a number and can be found on the front page of these collection instructions, on the collection tube or on the outside of the kit box. Use capital letters to register the kit code.

To register go to:

<https://questionnaire.mymicrozoo.com/medina>

Or scan the QR code on the right.

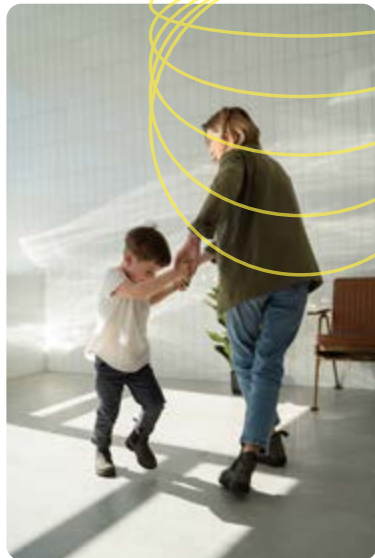
It is important that you register the kit. Without registration, we cannot compile the results.

Step 3: fill in the online questionnaire (takes 10 minutes)

Based on your microbiome, we can already say a lot about your (gut) health, but with more information we can help you even better. With the answers from the questionnaire, we can give you insights into which lifestyle and nutritional recommendations are important for you to improve your gut microbiome.


The questionnaire is not obligatory. If you prefer not to fill in a question, you can simply leave it blank. If you deliberately do not fill in the questions, your stool sample can still be analysed, but the results will be less personalised.

3. Collection instructions


Step 1: preparation

- | Take the box and its content to the toilet.
- | Place the collection tube upright.
- | Stick the feces collector to the toilet seat as indicated on its instructions.

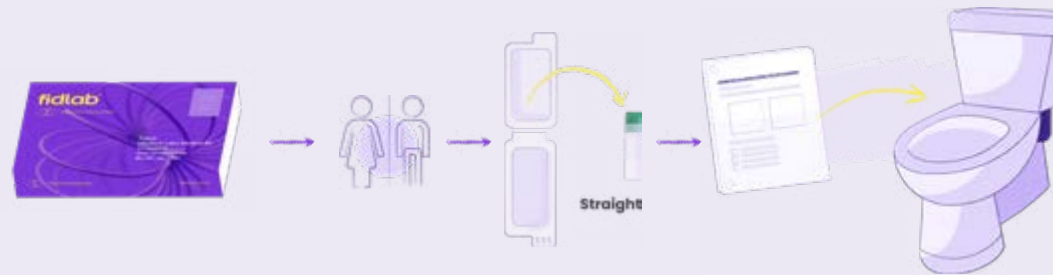Step 2: collection

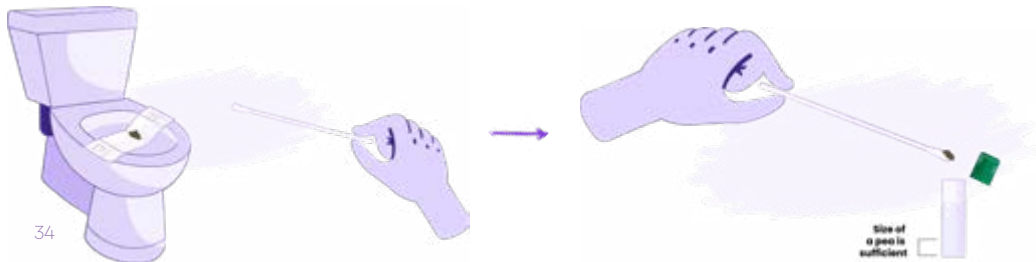- Leave your stool in the feces collector.
- Take the provided cotton swab from the foil.
- Scoop a little pea-sized stool onto the cotton swab.
- Using the cotton swab, place the stool into the collection tube. Do not break off the swab or leave it in the tube. You can reinsert the swab into the foil and throw it away with the household waste. Do not stuff the collection tube full with stool!
- Close the tube firmly with the cap to avoid leakage.
- Detach the feces collector and drop it into the toilet. After one minute, you can flush it.
- Wash your hands thoroughly after taking the sample.

Step 3: after collection

- Place the collection tube into the holder (tube protector in hard plastic).
- Tilt the holder at least 5 times to mix the fluid with the stool.
- Place the holder in the safety bag and put it in the kit box together with the collection instructions.
- Close the box and place it in the black shipping bag provided. You can now prepare the sample for shipment. Do this no later than 24 hours after sample collection.

4. Shipping

The return shipment to the laboratory has already been prepaid. You therefore do not have to pay any additional costs for this.

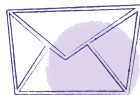
Step 1: register sample

Register your shipment via the online return page to receive a shipping label and to send your sample in for analysis.

To do so, go to:

<https://info887392.wixsite.com/returnmicrobiome/fidlab-retour>

Or scan the QR-code:

Step 2: your return

Fill in the requested details on the online return page to create and download your shipping label. You will receive the shipping instructions and your Track & Trace code via e-mail.

- Print the shipping label
- Attach the shipping label visibly to the black shipping bag
- Drop off your parcel at the nearest PostNL location free of charge
- Once your parcel has been delivered, you can track your return shipment

We recommend asking the PostNL employee for a deposit receipt. This is the proof of your return.

5. Results

Once your sample comes in, we will analyse the composition of your gut microbiome in 4 to 5 weeks.

We provide a report to your requesting healthcare provider. Contact them for discussion and follow-up.

6. Payment

You will receive an invoice from the Medina/ CMA lab after analysis and receipt of the result.

Unused kits cannot be taken back. If the lab has not received your sample for analysis, the full amount will be charged.

Questions?

If you have any questions regarding these instructions, please contact us:

 03 231 36 89

 info@fidlab.be

Please also consult the frequently asked questions around this sampling: www.fidlab.be/faq

fidlab.be/faq

www.fidlab.be

info@fidlab.be
03 231 36 89